



## STAT-Site<sup>®</sup> M<sup>Hgb</sup>

Power in the Palm  
of Your Hand—  
Truly Portable  
Hemoglobin Testing

CLIA'88  
WAIVED



STANBIO  
LABORATORY

The World's Source For Clinical Diagnostics

# STAT-Site® M<sup>Hgb</sup> Products

Reference No. 900900

## STAT-Site® M<sup>Hgb</sup> Meter

Includes STAT-Site® M<sup>Hgb</sup> meter, operating manual, quick reference guide and battery (extra battery also included)



Reference No. 901025

## STAT-Site® M<sup>Hgb</sup> Test Cards

Includes 100 test cards (4 packages of 25 each) and a lot-specific CODE key



Reference No. 900905

## STAT-Site® M<sup>Hgb</sup> Battery Pack

Includes 5 batteries (approximately 5,000 tests)

Reference No. 503000

## STAT-Site® Hemoglobin Controls

6 x 1.5 mL (3 low and 3 high level)

# STAT-Site® M<sup>Hgb</sup> Specs

Methodology _____	Reflectance photometry
Principle _____	Azidemethemoglobin
Calibration _____	Automatic (CODE Key)
Sample Type _____	Whole blood
Sample Size _____	1 drop (approximately 12 µL)
Linearity _____	6 to 21 g / dL
Within Run Precision CV _____	3.5% to 4.9%
Total Precision CV _____	2.9% to 4.2%

## Portable.

The **STAT-Site® M<sup>Hgb</sup>** is a battery operated, completely portable hemoglobin analyzer that fits in the palm of your hand. Finally, point-of-care testing right at your patient's side.

## Advanced.

The **STAT-Site® M<sup>Hgb</sup>** uses state-of-the-art electronics designed for optimum performance and reliability. Self-prompting graphics guide you through the simple test procedure. It even includes a "Last Result" feature that allows you to recall the last hemoglobin result.

## Simple.

Hemoglobin testing is quick and easy—reducing personnel training and improving performance. Unlike competing systems, the **STAT-Site® M<sup>Hgb</sup>** is virtually maintenance-free.

## Powerful.

With one single drop of fingerstick blood, the **STAT-Site® M<sup>Hgb</sup>** system provides precise hemoglobin analysis in seconds. Treatment can begin immediately, improving patient outcomes and lowering costs.

# Easy as one, two, three.

**Step 1 INSERT THE TEST CARD**

**Step 2 APPLY THE SAMPLE**

**Step 3 READ THE RESULT**

CE CLIA '88 WAIVED

CODE\*: 85018QW

\* Exemplar code provided. Refer to regional CMS fee schedule for relevant codes and reimbursement rates.



**STANBIO**  
LABORATORY

™ The World's Source For Clinical Diagnostics

- General & Special Chemistry
- Pregnancy
- Point-of-Care
- Rapid Tests
- Therapeutic Drug Monitoring
- Serology
- Occult Blood
- Urinalysis
- Clinical Instrumentation

Stanbio Laboratory  
1261 North Main Street  
Boerne, Texas 78006 USA

866.782.6246 (U.S. Toll Free)  
Tel: 830.249.0772  
Fax: 830.249.0851

email: stanbio@stanbio.com  
[www.stanbio.com](http://www.stanbio.com)



## DISTRIBUTED BY

RSS0003.03 DAR #: 11.0273 Effective Date: 4/19/11  
©2011 Stanbio Laboratory. Printed in USA.

U.S. Patent Nos. 5,104,619; 5,270,219 • Canadian Patent No. 1321945 • European Patent Nos. 0395680, 0512059. Additional patents pending; Canadian Patent Application No. 2,067,724; German Patent Application No. P2851635.7; Italian Patent Application No. 0395680.